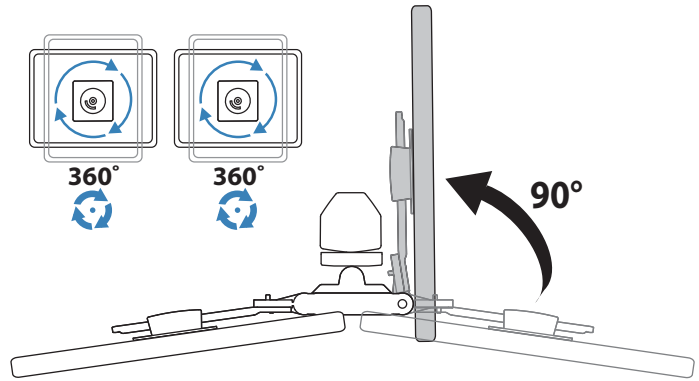
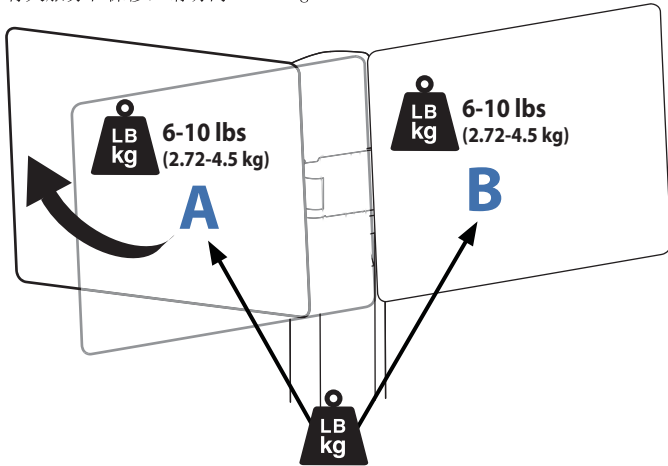
For service and warranty visit www.ergotron.com
 Si desea información de servicio y garantía, visite www.ergotron.com
 Pour toute question concernant la garantie ou le service client, consultez www.ergotron.com
 Weitere Informationen zu Service und Gewährleistung erhalten Sie unter www.ergotron.com.
 Ga voor service en garantie naar www.ergotron.com
 Per assistenza e la garanzia visitare www.ergotron.com
 Gå in på www.ergotron.com för service och garanti
 サービスおよび保証について詳しくは、www.ergotron.com をご覧ください。
 有关服务和保修，请访问 www.ergotron.com

A+B = 12-20 lbs (5.4-9.1 kg)*

- *Total Weight Capacity when mounted to the SV32 Carts with LiFe Power System is: A+B = 12-18 lbs (5.4-8.2 kg)
- *La capacidad de soporte de peso total montado en carros SV32 con sistema de alimentación LiFe es: A+B = 12-18 lbs (5.4-8.2 kg)
- *Capacité pondérale totale quand il est monté sur les chariots SV32 avec le système d'alimentation LiFe : A+B = 12-18 lbs (5.4-8.2 kg)
- *Tragkraft bei Montage auf SV32 Wagen mit LiFe-Stromversorgungssystem: A+B = 12-18 lbs (5.4-8.2 kg)
- *De totale gewichtscapaciteit bij bevestiging aan de SV32-wagens met het LiFe-stroomsysteem is: A+B = 12-18 lbs (5.4-8.2 kg)
- *La portata complessiva quando fissato ai carrelli SV32 con sistema di alimentazione LiFe è: A+B = 12-18 lbs (5.4-8.2 kg)
- *Total viktkapacitet när monterad till SV32-vagn med LiFe kraftsystem är: A+B = 12-18 lbs (5.4-8.2 kg)
- *LiFe電力システム付きSV32カートにマウントされているときの総重量： A+B = 12-18 lbs (5.4-8.2 kg)
- *安装到采用锂铁 (LiFe) 电源系统的SV32 推车时的总承重能力为： A+B = 12-18 lbs (5.4-8.2 kg)

! SV31/SV32 Carts:

CAUTION: If the combined LCD and CPU weight is greater than 23 lbs (10.4 kg) then the CPU must be mounted to the rear of the cart using the Universal CPU Holder accessory (ordered separately).

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! SV41/SV42 Carts:

CAUTION: If the combined LCD and CPU weight is greater than 27 lbs (12.2 kg) then the CPU must be mounted to the rear of the cart using the Universal CPU Holder accessory (ordered separately).

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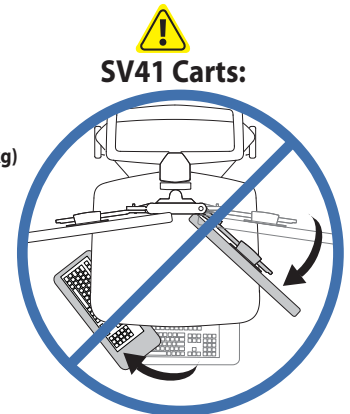
IMPORTANT! This product will need tension adjustments once installation is complete. Make sure all equipment is properly installed on the product before attempting range of motion or tension adjustments. Any time equipment is added or changed on this product resulting in a different mounted weight, you should repeat the adjustment steps to ensure safe and optimum operation. This product should move smoothly and easily through the full range of motion and stay where you set it. If movement is difficult or the product does not stay where you set it, follow the adjustment instructions to loosen or tighten the tension to create a smooth, easy motion. Depending on your product and the adjustment, it may take many turns to notice a difference.

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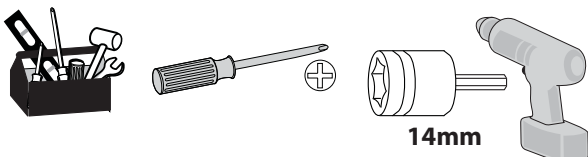
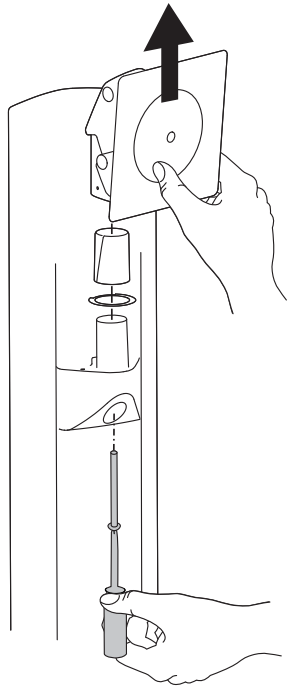
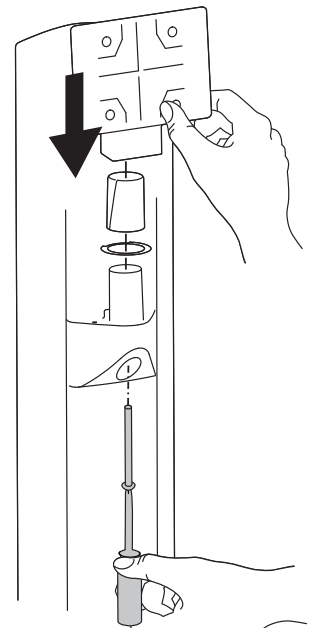
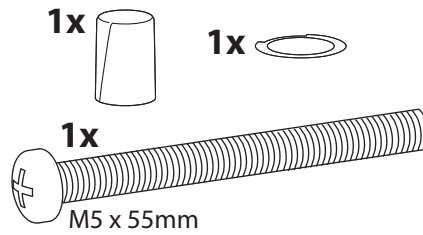
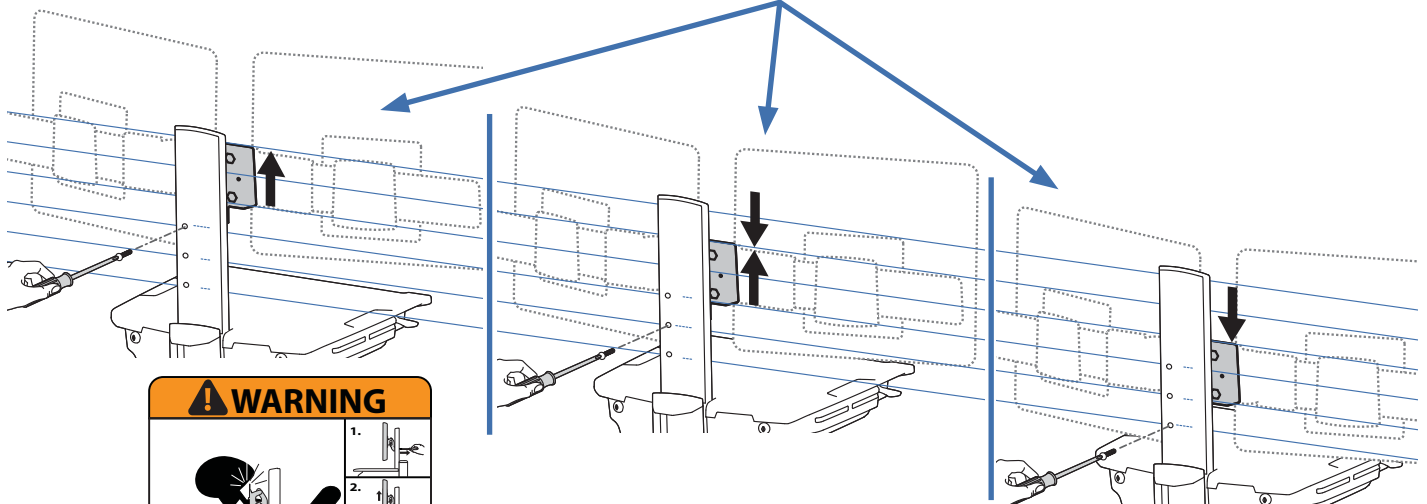
M4 x 8mm

⚠ WARNING

IMPACT HAZARD!
MOVING PARTS CAN CRUSH AND CUT.
DO NOT remove Stop Screw without monitor attached.
Doing so will cause monitor pivot to shoot up rapidly and may cause personal injury.

To Remove Monitor:

1. Remove stop screw while monitor is attached.
2. Lift monitor to highest position.
3. Remove monitor.

Failure to heed this warning may result in serious personal injury or property damage!

For More information and instructions refer to product guide at <http://4support.ergotron.com> or contact Ergotron Customer Care at 1-800-888-8458.

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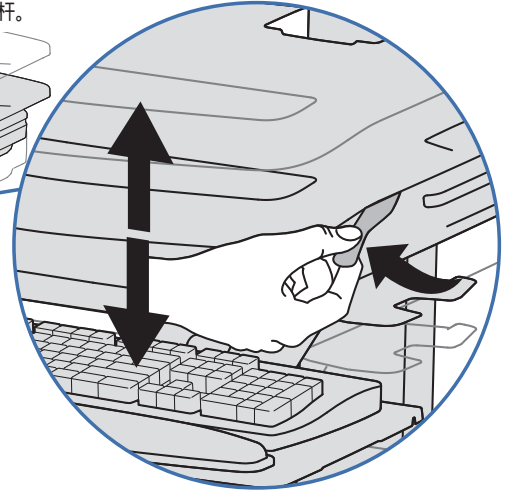
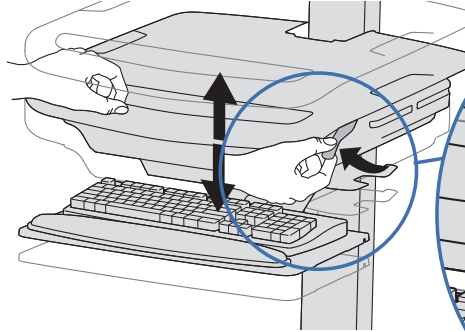
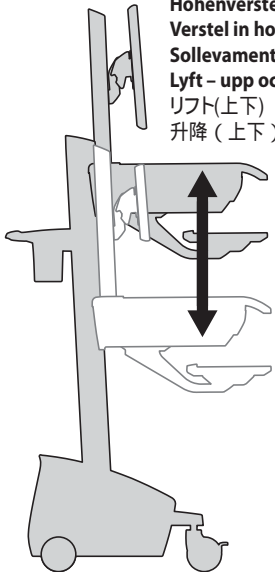
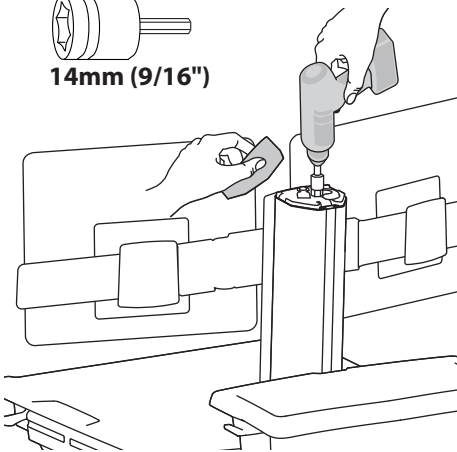
© 2012 Ergotron, Inc.
All rights reserved.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA. Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.

Americas Sales and Corporate Headquarters

St. Paul, MN USA
(800) 888-8458
+1-651-681-7600
www.ergotron.com
sales@ergotron.com

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