

Warning: Installation of this firmware update will reset your cart to its default factory settings.

- The Master PIN, all User PINs*, the Pharmacy PIN, and “alarm mute” setting will be reset to factory defaults.
- Any custom settings that have been applied to the cart will need to be reprogrammed.


* If your PINs are deployed through StyleLink Enterprise, User PINs should not need to be reprogrammed.

Installation Requirements


- Installation requires Microsoft Windows XP or later along with .NET Framework 4
- The computer used for firmware installation must have an active Internet connection.
- Download the firmware update (.zip) and extract its contents to a folder on your computer’s desktop

Step 1: Connect the drawer system to the computer via USB.

Step 2: Open the **SV43FirmwareTool.exe** file.

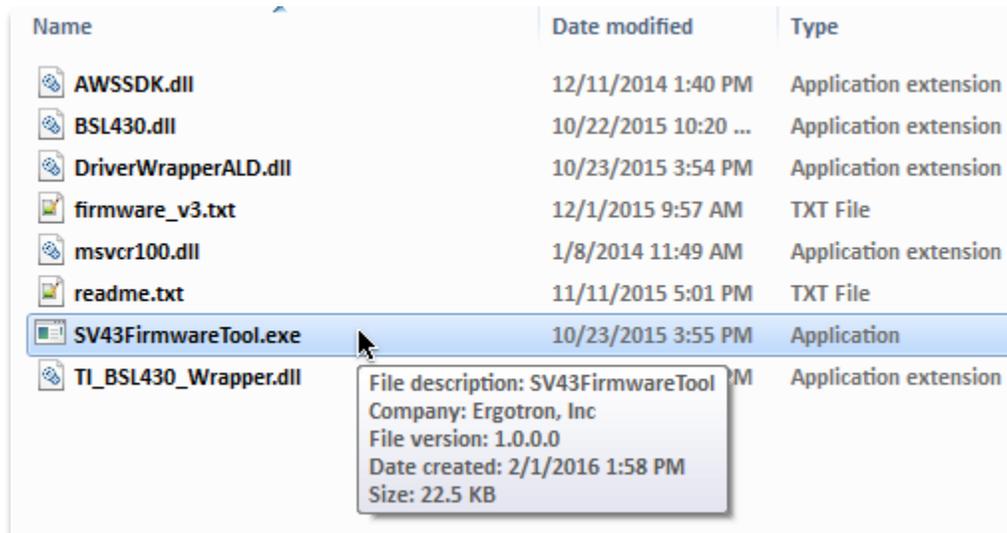

Step 3: Verify the status is “Connected” and click the “Upgrade Firmware” button.

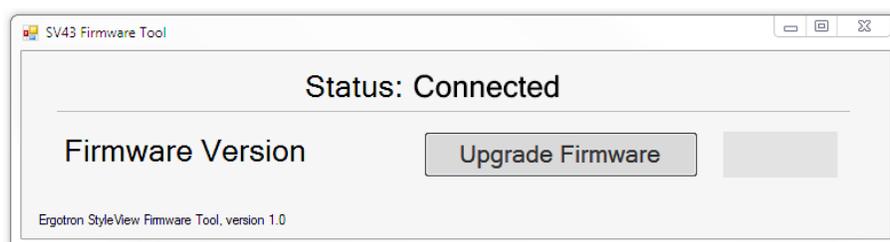

Important Do not disconnect the USB cable or close the application until installation is complete.