

## **Healthcare Cleaning Recommendations**

You can use most hospital-grade, diluted, non-abrasive, non-corrosive cleaning solutions on all plastic and metal surfaces, whether painted, polished or unfinished. See the following pages for a list of tested cleaning solutions by product family.

These are cleaning recommendations only and do not replace your cleaning and infection control requirements. Please consult your infection control administrator or epidemiologist for all cleaning and infection control procedures and processes.

### Helpful tips:

- Be sure to test any cleaning solution on a small, hidden area before widespread use.
- Pen and permanent and dry erase markers should be removed with 91% Isopropyl alcohol (generic) and a soft cloth.
- lodine stains can be removed with commonly used cleaners and a soft cloth.

#### For your safety and to protect your equipment:

- To avoid the risk of electric shock, do not expose electrical components to water, cleaning solutions or other potentially corrosive liquids or substances.
- Do not use flammable cleaners on cart surfaces due to the proximity to electrical power.
- Never use steel wool or other abrasive materials that will damage the surface finish.
- Do not use strong solvents such as trichloroethylene and acetone as they can damage the surface finish.
- Do not immerse the cart or cart components in liquid or allow liquids to flow into the cart. Wipe all cleaners off the surface immediately using a damp cloth and dry the surface after cleaning.



To support infection control, please clean, sanitize and sterilize all Ergotron equipment prior to first use and frequently while in use, in accordance with applicable health organization and government guidelines. This includes, but is not limited to, the World Health Organization, the Centers for Disease Control and Infection, the National Institutes of Health and healthcare industry guidelines. Guidelines are evolving, so we recommend checking these sites frequently.

Custom: custom@ergotron.com

#### TABLE OF CONTENTS CareFit™ Carts 3 Combos 5 **Enclosures** 7 StyleView® 9 Laptop, LCD Pivot, Pole Carts LCD Arm Carts 11 SV10 Carts 13 15 LCD Pivot Cart with Large Worksurface 17 Combos, Bright White Combos, Polished Aluminum 19 Sit-Stand Vertical Lifts 21 **Appendix** 23



## How to Use This Guide

This guide was developed to help customers understand the compatibility of select cleaning agents that have been specifically tested on Ergotron healthcare solutions. Please note, our internal testing focused on the cosmetic impact of the listed solutions and did not account for potential structural changes over time or their effectiveness in cleaning or disinfecting the products.

Active ingredients between similar cleaning agents may vary. Check the relevant manufacturer's website for the current active ingredients list, and be sure to test a small area of the product before applying a cleaning agent to the full solution. Cleaning solutions with similar ingredients listed in this guide may also be compatible. Conduct your own testing before use.

As always, follow the cleaning agent manufacturer's directions, and consult your infection control administrator or epidemiologist for all cleaning procedures and processes. We will update our guide as further testing is complete using additional cleaning agents and products.

#### KEY

#### Part numbers:

"X" placeholders represent various drawer configurations and global power options.

Example:

C52-12XX-X applies to C52-1201-1, as well as C52-1211-3.

## Product part colors:

Match the designated color for each product part with the product image to understand the tested area.

#### Compatible:

Internal Ergotron testing showed no cosmetic changes to product parts with selected cleaning agent.

#### May cause cosmetic changes to material:

Minimal cosmetic changes may become visible after prolonged use; other solutions are recommended.

#### Not compatible:

Internal Ergotron testing showed cosmetic changes after use; not recommended.

### Not tested:

The cleaning agent has not yet been tested on the specified Ergotron product part.

Contact your Ergotron representative or local Customer Care team with any questions about this information. Please refer to page 1 for contact details.



PRODUCT FAMILY: CAREFIT

PRODUCT TYPES: CARTS (WHITE/WARM GREY)

C50-1100-0 C50-3500-0 C50-3510-0 C52-12XX-X C52-22XX-X

	P	roduc	t Part	(if app	licabl	e)	
vrm / Tilt / Riser	Vorksurface / Head Unit	eyboard & Mouse Tray	landles	evers / Buttons	Drawers	Solumn	lase

Category	Cleaning Agent			Arm / Tilt	Worksurfa	Keyboarc	Handles	Levers / E	Drawers	Column	Base
Category	Brand	Туре		Ā	×	Ke	Ξ̈́	Le	مّ	ပိ	Ba
Alcohol	Isopropyl alcohol (generic), 99%	Liquid		<b>√</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>/</b>	<b>&lt;</b>	<b>✓</b>
Alcohol	Sterillium® Comfort Gel™, Hand Antiseptic	Liquid		<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Ammonia	Windex®, SC Johnson Professional®	Liquid		<b>√</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>/</b>	<b>/</b>
IPA, Quarternary Ammonium	Super Sani-Cloth®, PDI	Wipes		<b>√</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>
IFA, Qualternary Ammonium	Sani-Cloth® Plus	Wipes		<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>/</b>	<b>✓</b>
	CaviWipes™, Metrex	Wipes	ility	<b>√</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>
Quarternary Ammonium	Clinell® BCW200 Universal Wipes (green)	Wipes	Compatibility	<b>√</b>	<b>✓</b>	<b>/</b>	<b>✓</b>	<b>/</b>	<b>✓</b>	<b>✓</b>	<b>/</b>
	Clorox® Versasure® Wipes	Wipes	Product Co	*	<b>/</b>	<b>/</b>	<b>/</b>	*	<b>/</b>	<b>/</b>	*
Ethanol, Quarternary Ammonium	Lysol® Disinfectant Spray	Spray	Pro	<b>✓</b>	<b>✓</b>	<b>/</b>	<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>
Bleach	Clorox Healthcare® Bleach Germicidal Wipes	Wipes		<b>✓</b>	<b>✓</b>	<b>/</b>	<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>
	Diversey™ Oxivir® Tb Wipes, Virox Technologies	Wipes		<b>✓</b>	*	*	<b>✓</b>	<b>/</b>	*	<b>/</b>	<b>/</b>
Hydrogen Peroxide	Clorox Healthcare® Hydrogen Peroxide Wipes	Wipes		<b>√</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>/</b>	<b>/</b>
	Hydrogen Peroxide	Liquid		<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Peracetic	Clinell® Peracetic Acid Wipes CS25 (red)	Wipes		*	<b>/</b>	<b>/</b>	<b>/</b>	*	<b>/</b>	<b>/</b>	*

Compatible

May cause cosmetic changes to material

Not compatible

Not tested



# PRODUCT FAMILY: CAREFIT PRODUCT TYPES: COMBOS (WHITE)


45-618-251 45-619-251 45-621-251 45-622-251

Р	roduc	t Part	(if app	licabl	e)
				ack	
ser		rface	rd & Mouse Tray	ı <b>ck</b> < Covers hide the Wall Track	ick Covers

Category	Cleaning Agent			Tilt / Rise	Ε	Worksurfa	Keyboarc	Wall Tracl	Wall Trac
Odlegoly	Brand	Туре		Ē	Arm	š	Ke	Wa	Š
Alcohol	Isopropyl alcohol (generic), 99%	Liquid		<b>/</b>	<b>✓</b>	<b>/</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>
Alconor	Sterillium® Comfort Gel™, Hand Antiseptic	Liquid		<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>/</b>	<b>/</b>
Ammonia	Windex®, SC Johnson Professional®	Liquid		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>
IPA, Quarternary Ammonium	Super Sani-Cloth®, PDI	Wipes		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>/</b>
ii A, Quarternary Ammonium	Sani-Cloth® Plus	Wipes		<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>/</b>
	CaviWipes™, Metrex	Wipes	ility	<b>/</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>
Quarternary Ammonium	Clinell® BCW200 Universal Wipes (green)	Wipes	Compatibility	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>
	Clorox® Versasure® Wipes	Wipes	Product Co	<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>
Ethanol, Quarternary Ammonium	Lysol® Disinfectant Spray	Spray	Pre	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>
Bleach	Clorox Healthcare® Bleach Germicidal Wipes	Wipes		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>
	Diversey™ Oxivir® Tb Wipes, Virox Technologies	Wipes		*	*	*	*	*	*
Hydrogen Peroxide	Clorox Healthcare® Hydrogen Peroxide Wipes	Wipes		<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>
	Hydrogen Peroxide	Liquid		<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>
Peracetic	Clinell® Peracetic Acid Wipes CS25 (red)	Wipes		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>





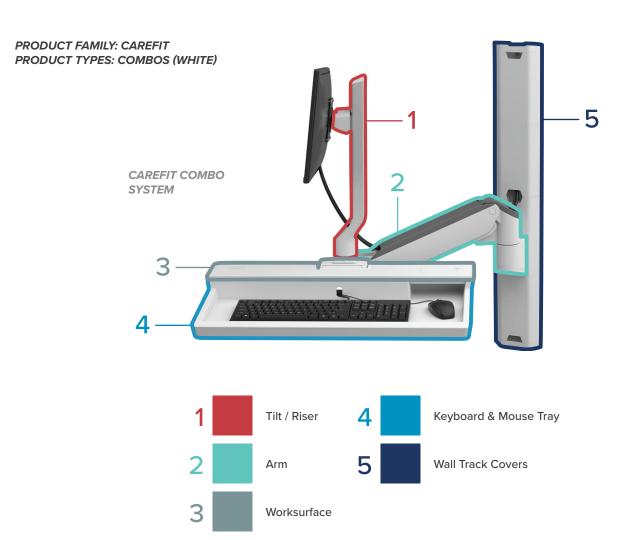

PRODUCT FAMILY: CAREFIT


PRODUCT TYPES: ENCLOSURES (BLACK OR WHITE)


61-367-060									
61-367-030  Category	Cleaning Agent Brand	Туре		Lift Engine	Handles	Worksurface	Keyboard & Mouse Tray	Covers	Wall Covers
	Isopropyl alcohol (generic), 99%	Liquid		. /	. /	. /	. /	. /	./
Alcohol		Liquid		<b>V</b>	<b>V</b>	<b>V</b>	<b>V</b>	<b>V</b>	<b>V</b>
	Sterillium® Comfort Gel™, Hand Antiseptic	Liquid		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Ammonia	Windex®, SC Johnson Professional®	Liquid		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>✓</b>
IDA Outstand Autstanding	Super Sani-Cloth®, PDI	Wipes		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
IPA, Quarternary Ammonium	Sani-Cloth® Plus	Wipes		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
	CaviWipes™, Metrex	Wipes	ility	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Quarternary Ammonium	Clinell® BCW200 Universal Wipes (green)	Wipes	mpatib	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
	Clorox® Versasure® Wipes	Wipes	Product Compatibility	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Ethanol, Quarternary Ammonium	Lysol® Disinfectant Spray	Spray	Pro	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Bleach	Clorox Healthcare® Bleach Germicidal Wipes	Wipes		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
	Diversey <sup>™</sup> Oxivir <sup>®</sup> Tb Wipes, Virox Technologies	Wipes		<b>/</b>	*	*	*	<b>/</b>	
Hydrogen Peroxide	Clorox Healthcare® Hydrogen Peroxide Wipes	Wipes		<b>&lt;</b>	<b>&lt;</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>✓</b>
	Hydrogen Peroxide	Liquid		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Peracetic	Clinell® Peracetic Acid Wipes CS25 (red)	Wipes		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>



Product Part (if applicable)



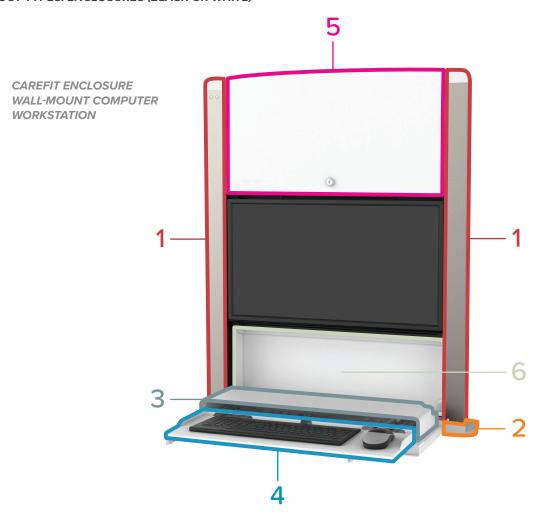

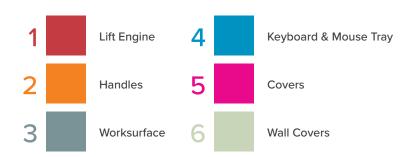

## PRODUCT FAMILY: STYLEVIEW

PRODUCT TYPES: CARTS (LAPTOP, LCD PIVOT, POLE — WHITE)

SV40-6100-0SV42-8302-1SV40-6300-0SV43-11XX-XSV41-6100-0SV43-13XX-XSV41-6300-0SV44-11XX-XSV42-6102-XSV44-13XX-XSV42-6302-X24-818-211

	Pi	roduc	t Part	(if app	licabl	e)	
Arm / Tilt / Riser	Worksurface / Head Unit	Keyboard & Mouse Tray	Handles	Levers / Buttons	Drawers	Column	Base

Category	Cleaning Agent			Arm / Tilt	Worksurfa	Keyboarc	Handles	Levers /	Drawers	Column	Base
<b>3</b> ,	Brand	Туре		Ā	Š	Ä	坣	Le	۵	Ö	B
	Isopropyl alcohol (generic), 99%	Liquid		<b>\</b>	<b>✓</b>	<b>√</b>	<b>✓</b>	*	<b>✓</b>	<b>✓</b>	*
Alcohol	Sterillium® Comfort Gel™, Hand Antiseptic	Liquid		<b>✓</b>	<b>✓</b>	<b>/</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>
Ammonia	Windex®, SC Johnson Professional®	Liquid		<b>✓</b>	<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>/</b>	<b>/</b>
IPA, Quarternary Ammonium	Super Sani-Cloth®, PDI	Wipes		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	*	<b>/</b>	<b>/</b>	*
IPA, Quarternary Ammonium	Sani-Cloth® Plus	Wipes		<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	*	<b>/</b>	<b>/</b>	*
	CaviWipes™, Metrex	Wipes	lity	<b>/</b>	<b>/</b>	<b>\</b>	<b>/</b>	*	<b>/</b>	<b>/</b>	*
Quarternary Ammonium	Clinell® BCW200 Universal Wipes (green)	Wipes	Compatibility	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	*	<b>✓</b>	<b>✓</b>	*
	Clorox® Versasure® Wipes	Wipes	Product Co	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	*	<b>✓</b>	<b>✓</b>	*
Ethanol, Quarternary Ammonium	Lysol® Disinfectant Spray	Spray	Pro	<b>/</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>/</b>	<b>✓</b>	<b>✓</b>	<b>/</b>
Bleach	Clorox Healthcare® Bleach Germicidal Wipes	Wipes		<b>/</b>	<b>✓</b>	<b>/</b>	<b>✓</b>	*	<b>✓</b>	<b>✓</b>	*
	Diversey™ Oxivir® Tb Wipes, Virox Technologies	Wipes								<b>√</b>	
Hydrogen Peroxide	Clorox Healthcare® Hydrogen Peroxide Wipes	Wipes		<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	*	<b>✓</b>	<b>✓</b>	*
	Hydrogen Peroxide	Liquid		<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>/</b>
Peracetic	Clinell® Peracetic Acid Wipes CS25 (red)	Wipes		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	*	<b>/</b>	<b>/</b>	*


Compatible

May cause cosmetic changes to material

Not compatible

Not tested



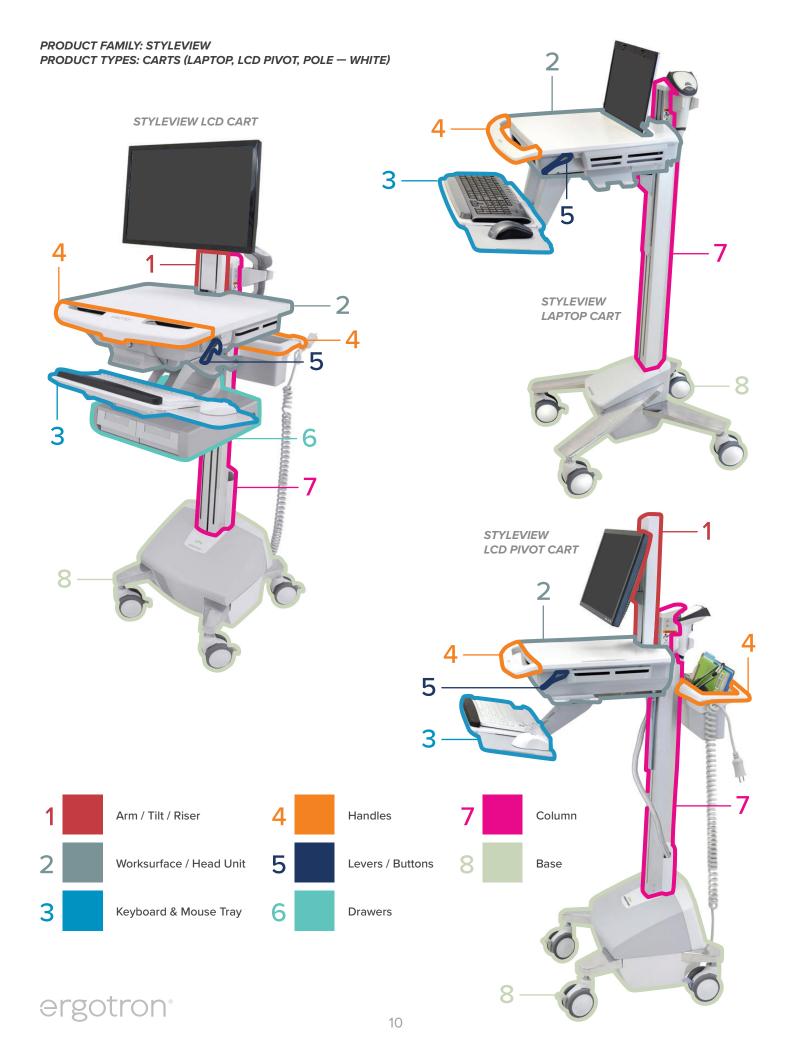

PRODUCT FAMILY: STYLEVIEW

PRODUCT TYPES: CARTS (LCD ARM — WHITE)

SV41-6200-0 SV42-6202-X SV43-12XX-X SV44-12XX-X

	P	roduc	t Part	(if app	licabl	e)	
Arm / Tilt / Riser	Worksurface / Head Unit	Keyboard & Mouse Tray	Handles	Levers / Buttons	Drawers	Column	Base

Cotomoni	Cleaning Agent			Arm / Tilt	Worksurf	Keyboard	Handles	Levers /	Drawers	Column	e Se
Category	Brand	Туре		Arr	No.	Ke	Ha	Le	Dr	ပိ	Base
Alexhal	Isopropyl alcohol (generic), 99%	Liquid		*	<b>✓</b>	<b>✓</b>	<b>✓</b>	*	<b>/</b>	<b>✓</b>	*
Alcohol	Sterillium® Comfort Gel™, Hand Antiseptic	Liquid		<b>✓</b>	<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>/</b>
Ammonia	Windex®, SC Johnson Professional®	Liquid		<b>✓</b>	<b>/</b>						
IPA, Quarternary Ammonium	Super Sani-Cloth®, PDI	Wipes		*	<b>/</b>	<b>/</b>	<b>/</b>	*	<b>/</b>	<b>/</b>	*
IFA, Quarternary Ammonium	Sani-Cloth® Plus	Wipes		*	<b>✓</b>	<b>✓</b>	<b>/</b>	*	<b>/</b>	<b>/</b>	*
	CaviWipes™, Metrex	Wipes	lity	*	<b>/</b>	<b>/</b>	<b>/</b>	*	<b>/</b>	<b>/</b>	*
Quarternary Ammonium	Clinell® BCW200 Universal Wipes (green)	Wipes	Compatibility	*	<b>/</b>	<b>✓</b>	<b>✓</b>	*	<b>/</b>	<b>✓</b>	*
	Clorox® Versasure® Wipes	Wipes	Product Co	*	<b>✓</b>	<b>✓</b>	<b>✓</b>	*	<b>✓</b>	<b>✓</b>	*
Ethanol, Quarternary Ammonium	Lysol® Disinfectant Spray	Spray	Pro	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>
Bleach	Clorox Healthcare® Bleach Germicidal Wipes	Wipes		*	<b>/</b>	<b>/</b>	<b>✓</b>	*	<b>/</b>	<b>/</b>	*
	Diversey™ Oxivir® Tb Wipes, Virox Technologies	Wipes								<b>/</b>	
Hydrogen Peroxide	Clorox Healthcare® Hydrogen Peroxide Wipes	Wipes		*	<b>✓</b>	<b>✓</b>	<b>✓</b>	*	<b>✓</b>	<b>✓</b>	*
	Hydrogen Peroxide	Liquid		<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>
Peracetic	Clinell® Peracetic Acid Wipes CS25 (red)	Wipes		*	<b>✓</b>	<b>/</b>	<b>/</b>	*	<b>/</b>	<b>✓</b>	*


Compatible

May cause cosmetic changes to material

Not compatible

Not tested




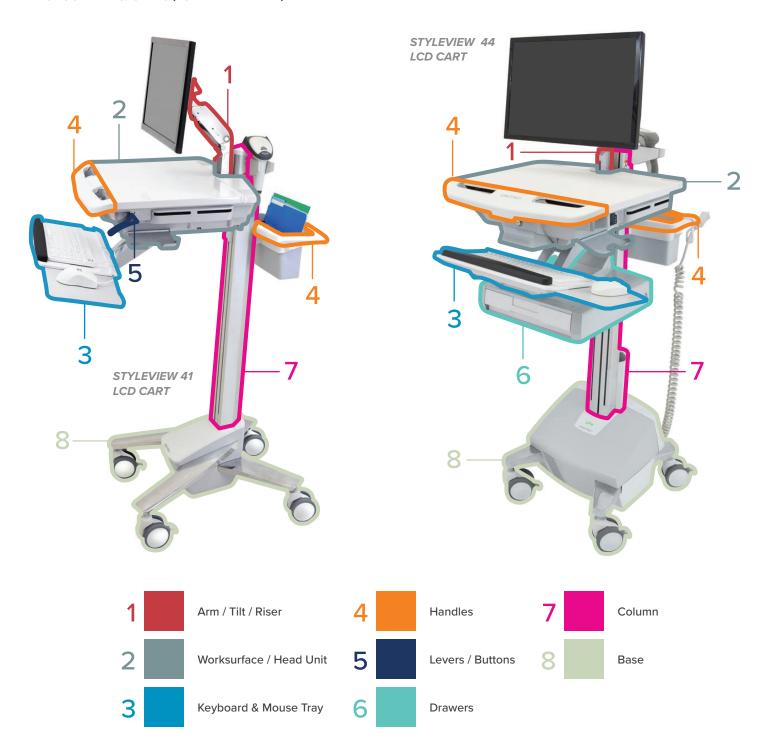

PRODUCT FAMILY: STYLEVIEW PRODUCT TYPES: CARTS (WHITE)

SV10-1100-0 SV10-1400-0

SV10-1400-0 SV10-1800-0									
				Arm / Tilt / Riser	Worksurface / Head Unit	S	Levers / Buttons	_	
Category	Cleaning Agent			Е/ Ш	orksı	Handles	vers	Column	Base
	Brand	Туре		Ā	š	Ϋ́	Le	ပိ	Ba
Alcohol	Isopropyl alcohol (generic), 99%	Liquid		<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Alcohor	Sterillium® Comfort Gel™, Hand Antiseptic	Liquid		<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Ammonia	Windex®, SC Johnson Professional®	Liquid		<b>✓</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>
IPA, Quarternary Ammonium	Super Sani-Cloth®, PDI	Wipes		<b>/</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>✓</b>
ira, qualternaly animonium	Sani-Cloth® Plus	Wipes		<b>/</b>		<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
	CaviWipes™, Metrex	Wipes	ility	<b>/</b>	<b>✓</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Quarternary Ammonium	Clinell® BCW200 Universal Wipes (green)	Wipes	mpatip	<b>✓</b>		<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
	Clorox® Versasure® Wipes	Wipes	Product Compatibility	<b>✓</b>		<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Ethanol, Quarternary Ammonium	Lysol® Disinfectant Spray	Spray	Pro	<b>/</b>		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>
Bleach	Clorox Healthcare® Bleach Germicidal Wipes	Wipes		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
	Diversey™ Oxivir® Tb Wipes, Virox Technologies	Wipes			<b>✓</b>			<b>/</b>	
Hydrogen Peroxide	Clorox Healthcare® Hydrogen Peroxide Wipes	Wipes		<b>/</b>	<b>✓</b>	<b>/</b>	<b>✓</b>	<b>✓</b>	<b>/</b>
	Hydrogen Peroxide	Liquid		<b>✓</b>		<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Peracetic	Clinell® Peracetic Acid Wipes CS25 (red)	Wipes		<b>/</b>		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>

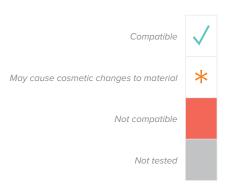


**Product Part (if applicable)** 




PRODUCT FAMILY: STYLEVIEW PRODUCT TYPES: CARTS (WHITE)



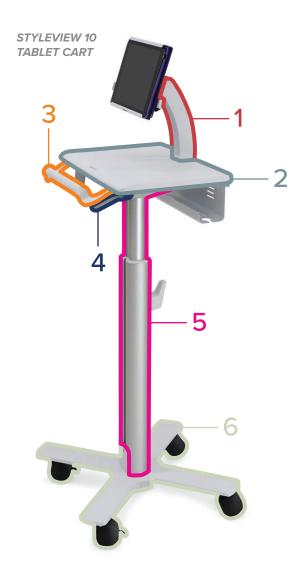




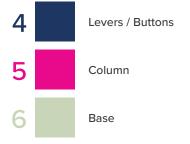

PRODUCT FAMILY: STYLEVIEW
PRODUCT TYPES: LCD PIVOT CART WITH LARGE WORKSURFACE
(WHITE)

SV41-6320


	Cleaning Agent				Worksurface / Hea	Handles	Levers / Buttons	Column	Φ
Category	Brand	Туре		Arm / Tilt / Riser	Wor	Han	Lev	Coll	Base
Alaahal	Isopropyl alcohol (generic), 99%	Liquid		<b>/</b>		<b>/</b>	*	<b>✓</b>	*
Alcohol	Sterillium® Comfort Gel™, Hand Antiseptic	Liquid		<b>✓</b>		<b>/</b>	<b>/</b>	<b>✓</b>	<b>✓</b>
Ammonia	Windex®, SC Johnson Professional®	Liquid		<b>/</b>		<b>/</b>	<b>&lt;</b>	<b>/</b>	<b>✓</b>
IDA Quartarnam Ammanium	Super Sani-Cloth®, PDI	Wipes		<b>/</b>		<b>/</b>	*	<b>/</b>	*
IPA, Quarternary Ammonium	Sani-Cloth® Plus	Wipes		<b>/</b>		<b>/</b>	*	<b>✓</b>	*
	CaviWipes™, Metrex	Wipes	lity	<b>/</b>		<b>/</b>	*	<b>/</b>	*
Quarternary Ammonium	Clinell® BCW200 Universal Wipes (green)	Wipes	mpatibi	<b>/</b>		<b>/</b>	*	<b>/</b>	*
	Clorox® Versasure® Wipes	Wipes	Product Compatibility	<b>/</b>		<b>/</b>	*	<b>✓</b>	*
Ethanol, Quarternary Ammonium	Lysol® Disinfectant Spray	Spray	Pro	<b>/</b>		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>
Bleach	Clorox Healthcare® Bleach Germicidal Wipes	Wipes		<b>/</b>		<b>/</b>	*	<b>/</b>	*
	Diversey™ Oxivir® Tb Wipes, Virox Technologies	Wipes						<b>✓</b>	
Hydrogen Peroxide	Clorox Healthcare® Hydrogen Peroxide Wipes	Wipes		<b>/</b>		<b>/</b>	*	<b>✓</b>	*
	Hydrogen Peroxide	Liquid		<b>/</b>		<b>/</b>	<b>/</b>	<b>✓</b>	<b>✓</b>
Peracetic	ClineII® Peracetic Acid Wipes CS25 (red)	Wipes		<b>/</b>		<b>/</b>	*	<b>/</b>	*

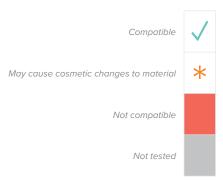

**Product Part (if applicable)** 

/ Head Unit



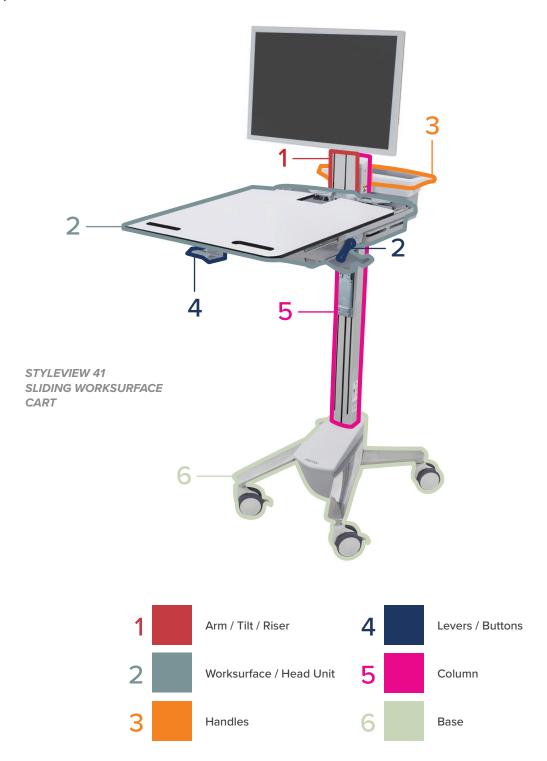


16

### PRODUCT FAMILY: STYLEVIEW PRODUCT TYPES: COMBOS (WHITE)


45-583-216 45-272-216 45-270-216 45-260-216 45-266-216 45-273-216 45-594-216 45-271-216

45-260-216			ı	Riser		Worksurface	Keyboard & Mouse Tray	Wall Track	Wall Track Covers
Category	Cleaning Agent Brand	Туре		Tilt / Riser	Arm	Work	Keybo	Wall 1	Wall 1
	Isopropyl alcohol (generic), 99%	Liquid		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Alcohol	Sterillium® Comfort Gel™, Hand Antiseptic	Liquid		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Ammonia	Windex®, SC Johnson Professional®	Liquid		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
IDA Outstander Australia	Super Sani-Cloth®, PDI	Wipes		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
IPA, Quarternary Ammonium	Sani-Cloth® Plus	Wipes		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
	CaviWipes™, Metrex	Wipes	ility	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Quarternary Ammonium	Clinell® BCW200 Universal Wipes (green)	Wipes	Product Compatibility	<b>✓</b>	<b>✓</b>	<b>/</b>	<b>✓</b>	<b>/</b>	<b>✓</b>
	Clorox® Versasure® Wipes	Wipes	duct Co	<b>✓</b>	<b>✓</b>	<b>✓</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Ethanol, Quarternary Ammonium	Lysol® Disinfectant Spray	Spray	Pro	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Bleach	Clorox Healthcare® Bleach Germicidal Wipes	Wipes		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
	Diversey™ Oxivir® Tb Wipes, Virox Technologies	Wipes		*	<b>/</b>	*	*	<b>/</b>	<b>✓</b>
Hydrogen Peroxide	Clorox Healthcare® Hydrogen Peroxide Wipes	Wipes		<b>/</b>	<b>/</b>	<b>✓</b>	<b>✓</b>	<b>/</b>	<b>✓</b>
	Hydrogen Peroxide	Liquid		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>
Peracetic	Clinell® Peracetic Acid Wipes CS25 (red)	Wipes		<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>√</b>

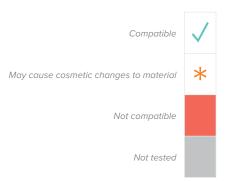

**Product Part (if applicable)** 



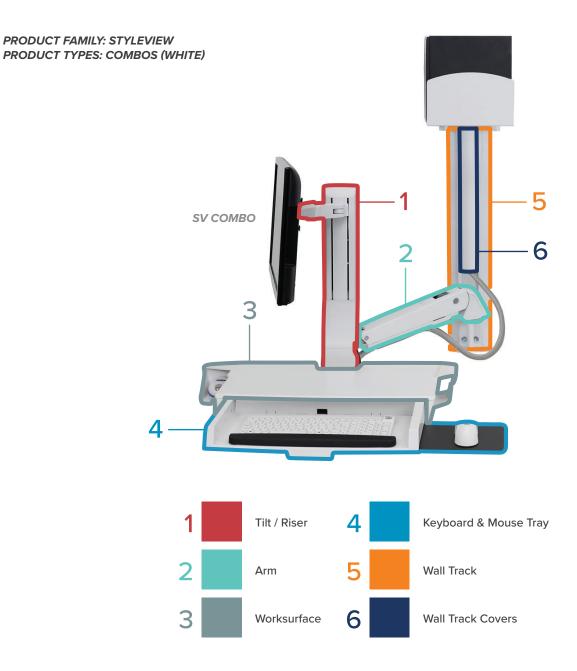

## PRODUCT FAMILY: STYLEVIEW

PRODUCT TYPES: COMBOS (POLISHED ALUMINUM)


45-583-026 45-272-026 45-270-026 45-260-026 45-266-026 45-273-026 45-594-026 45-271-026

45-260-026			ser		Worksurface	Keyboard & Mouse Tray	ack	Wall Track Covers	
Category	Cleaning Agent			Tilt / Riser	Arm	/orksu	eyboa	Wall Track	/all Tra
	Brand	Туре		-	⋖	>		>	<u> </u>
Alcohol	Isopropyl alcohol (generic), 99%	Liquid	<b>√</b>		*	<b>/</b>	<b>/</b>	*	<b>/</b>
Alconor	Sterillium® Comfort Gel™, Hand Antiseptic			<b>✓</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>/</b>	<b>/</b>
Ammonia	Windex®, SC Johnson Professional® Liquid		√ √ √	<b>/</b>	<b>/</b>	<b>✓</b>	<b>/</b>	<b>/</b>	
IDA Outstand Autstanding	Super Sani-Cloth®, PDI Wipe			*	<b>/</b>	<b>/</b>	*	<b>/</b>	
IPA, Quarternary Ammonium	Sani-Cloth® Plus Wipes			<b>/</b>	*	<b>✓</b>	<b>✓</b>	*	<b>✓</b>
Quarternary Ammonium	CaviWipes™, Metrex	riWipes™, Metrex Wipes		<b>/</b>	*	<b>/</b>	<b>/</b>	*	<b>/</b>
	Clinell® BCW200 Universal Wipes (green)		mpatibi	<b>/</b>	*	<b>/</b>	<b>✓</b>	*	<b>/</b>
	Clorox® Versasure® Wipes Wipes		Product Compatibility	<b>/</b>	*	<b>/</b>	<b>/</b>	*	<b>/</b>
Ethanol, Quarternary Ammonium	I VSOI® I JISINTACTANT SNraV		Pro	<b>/</b>	<b>/</b>	<b>/</b>	<b>/</b>	<b>✓</b>	<b>/</b>
Bleach	Clorox Healthcare® Bleach Germicidal Wipes	Wipes	<b>/</b>	*	<b>✓</b>	<b>/</b>	*	<b>/</b>	
	Diversey™ Oxivir® Tb Wipes, Virox Technologies	Wipes							<b>✓</b>
Hydrogen Peroxide	Clorox Healthcare® Hydrogen Peroxide Wipes			<b>/</b>	*	<b>/</b>	<b>/</b>	*	<b>/</b>
	Hydrogen Peroxide	Liquid		<b>/</b>	<b>✓</b>	<b>✓</b>	<b>/</b>	<b>✓</b>	<b>✓</b>
Peracetic	tic Clinell® Peracetic Acid Wipes CS25 (red) Wipes			<b>/</b>	*	<b>/</b>	<b>/</b>	*	<b>/</b>



**Product Part (if applicable)** 



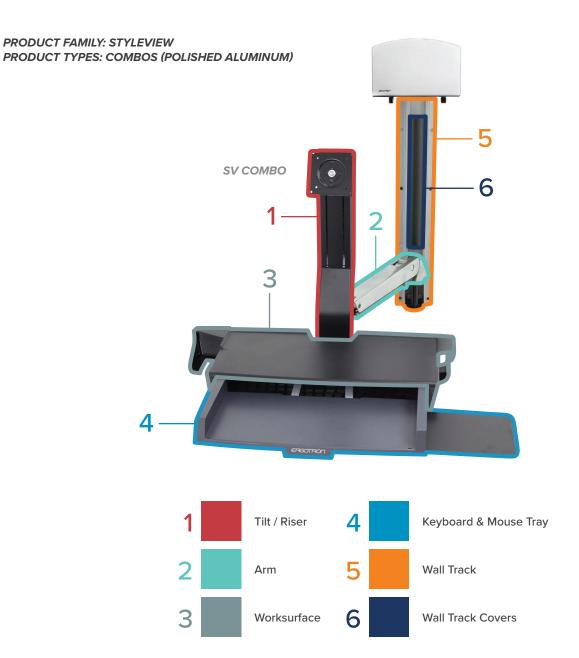


# PRODUCT FAMILY: STYLEVIEW PRODUCT TYPES: SIT-STAND VERTICAL LIFTS (WHITE OR BLACK)

61-080-062 61-080-085 61-081-062 61-081-085


Product Part (if applicable)					
jine	S	ırface	ard & Mouse Tray		


Category	Cleaning Agent			Lift Engir	Handles	Worksurf	Keyboar
Category	Brand	Туре		Lift	Ha	× ×	Ke
Alcohol	Isopropyl alcohol (generic), 99%	Liquid		<b>✓</b>	<b>✓</b>		<b>✓</b>
Alconor	Sterillium® Comfort Gel™, Hand Antiseptic		<b>✓</b>	<b>/</b>		<b>✓</b>	
Ammonia	Windex®, SC Johnson Professional®	Liquid		<b>✓</b>	<b>✓</b>		<b>/</b>
IPA, Quarternary Ammonium	Super Sani-Cloth®, PDI W			<b>/</b>	<b>/</b>		<b>/</b>
IPA, Quarternary Ammonium	Sani-Cloth® Plus Wipes				<b>/</b>		<b>/</b>
Quarternary Ammonium	CaviWipes™, Metrex	Wipes	ility	<b>/</b>	<b>/</b>		<b>/</b>
	Clinell® BCW200 Universal Wipes (green)	Product Compatibility	<b>✓</b>	<b>/</b>		<b>✓</b>	
	Clorox® Versasure® Wipes	Wipes	oduct Co	<b>✓</b>	<b>/</b>		<b>✓</b>
Ethanol, Quarternary Ammonium	Lysol® Disinfectant Spray		Pro	<b>/</b>	<b>/</b>		<b>/</b>
Bleach	Clorox Healthcare® Bleach Germicidal Wipes			<b>✓</b>	<b>/</b>		<b>/</b>
Hydrogen Peroxide	Diversey™ Oxivir® Tb Wipes, Virox Technologies	Wipes		*	*		*
	Clorox Healthcare® Hydrogen Peroxide Wipes	Wipes		<b>/</b>	<b>/</b>		<b>/</b>
	Hydrogen Peroxide	Liquid		<b>✓</b>	<b>/</b>		<b>/</b>
Peracetic	Clinell® Peracetic Acid Wipes CS25 (red) Wipes			<b>/</b>	<b>/</b>		<b>/</b>

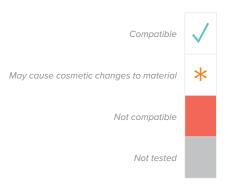

## STYLEVIEW SIT-STAND VERTICAL LIFT HIGH-TRAFFIC AREAS

STYLEVIEW SIT-STAND VERTICAL LIFT PATIENT ROOM

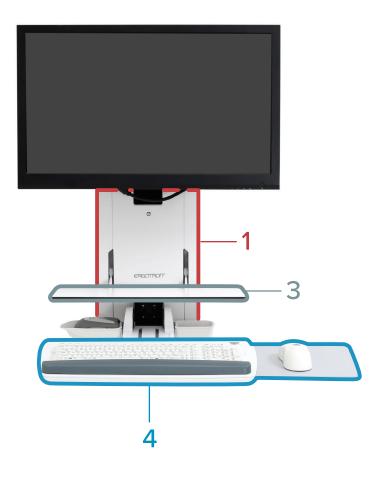

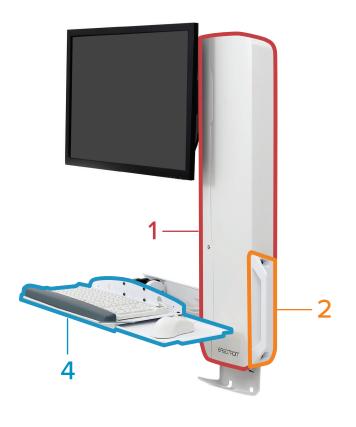



## **Appendix**

The active ingredients for each cleaning agent at the time of testing are listed below. Active ingredients may change, so be sure to check the relevant manufacturer's website for the current active ingredients list.

Cleaning Agent		Active Ingredients				
Brand Type		Active Ingredients				
Isopropyl alcohol (generic), 99%	Liquid	Isopropyl alcohol 99%				
Sterillium® Comfort Gel™, Hand Antiseptic	Liquid	Ethyl alcohol 85%				
Windex®, SC Johnson Professional®	Liquid	Ammonium Hydroxide, Isopropanolamine, 2-Hexoxyethanol				
Super Sani-Cloth®, PDI	Wipes	Isopropyl alcohol 55.5%, Quaternary ammonium compounds, C12-18-alkyl [(ethylphenyl) methyl] dimethyl, chlorides 0.25%, Alkyl Dimethyl Benzyl Ammonium Chloride 0.25%				
Sani-Cloth® Plus	Wipes	Isopropanol 10–20%, 2-Butoxyethanol < 5%, Quaternary ammonium compounds, C12–18-alkyl [(ethylphenyl) methyl] dimethyl, chlorides 0.125%, Benzyl-C12–18-alkyldimethyl ammonium chlorides 0.125%				
CaviWipes™, Metrex	Wipes	Propan-2-ol, isopropyl alcohol, isopropanol 17.20%, 2-Butoxyethanol, ethylene glycol monobutyl ether, butyl cellosolve 1–5%, Benzethonium Chloride (QA) 0.28%				
Clinell® BCW200 Universal Wipes (green)	Wipes	2-Phenoxyethanol 0.6%, Benzalkonium chloride 0.6%, Didecyldimethyl ammonium chloride 0.6%, Biphenyl-2-ol 0.1%				
Clorox®Versasure® Wipes	Wipes	0.25% n-Alkyl (C14, 60%; C16, 30%; C12, 5%; C18, 5%) Dimethyl Benzyl Ammonium Chloride, 0.25% n-Alkyl (C12, 68%; C14, 32%) Dimethyl Ethylbenzyl Ammonium Chloride				
Lysol® Disinfectant Spray	Spray	Ethanol, Alkyl (50% C14, 40% C12, 10% C16) Dimethyl benzyl ammonium saccharinate				
Clorox Healthcare® Bleach Germicidal Wipes	Wipes	Sodium hypochlorite NaOCI 0.55%				
Diversey™ Oxivir® Tb Wipes, Virox Technologies Wip		Hydrogen peroxide 0.5%				
Clorox Healthcare® Hydrogen Peroxide Wipes	Wipes	Hydrogen peroxide H2O2 1.4%				
Hydrogen Peroxide	Liquid	Hydrogen peroxide 3%				
Clinell® Peracetic Acid Wipes CS25 (red)	Wipes	Sodium Percarbonate ≤50%, Citric Acid ≤15%, Tetraacetylethylenediamine ≤ 25%				

